

C&M Export Instructions - iTero Only

EXPORTING STL FILES FROM ITERO or Intra Oral Scanner

Important Notes

- Make sure that you have captured the bite scans in correct occlusion. An accurate wax up and planning cannot be done if the bite is not captured correctly. Review scan to check occlusion.
- Make sure that there are no holes or distortion on the teeth. Holes or distortion may result in a guide that does not fit.

1. Log in to www.myitero.com with your practice login and password
2. Search for the patient name
3. Click on the patient name to open the case file
4. Choose Export
5. Choose the correct 3 options shown in the image below (Open shell, file per arch oriented in occlusion, STL + PLY)



Orders

ID	Scan Date	Case Type	Status
111019404	09/11/2022	Invisalign + iRecord	Completed

View Rx Change Case Type Viewer **Export** iTero Scan Report Invisalign Outcome Simulator

Invisalign Progress Assessment Invisalign Go Case Assessment Invisalign Go Outcome Simulator OrthoCAD



Export Type Open Shell

Data Format File per Arch (arches oriented in occl...

File Type STL+PLY

Hide patient name

Cancel Export



UPLOADING THE CASE TO C&M FOR SURGICAL REVIEW

1. Fill out the [C&M upload form](#) on the <https://www.cmsurgicalguides.com/> website
2. Wax Ups are encouraged for accurate planning. Wax Ups are \$25 per tooth. Be sure to include wax up sites for missing teeth if needed
3. Choose one of the following options

WAX-UP: Virtual Wax-Up - CBCT + Intraoral Scan (indicate #s)


NON WAX-UP: Merge DICOM & STL (no wax-up)

4. Browse to the download folder and attach the Orthocad export folder that you exported from www.myitero.com (see page 1 for export instructions)

Please zip DICOM & STL files into one folder and use the patient name as the title of the folder*

Drag and drop files here

5. Make sure to select the one with the date and time that you exported the file

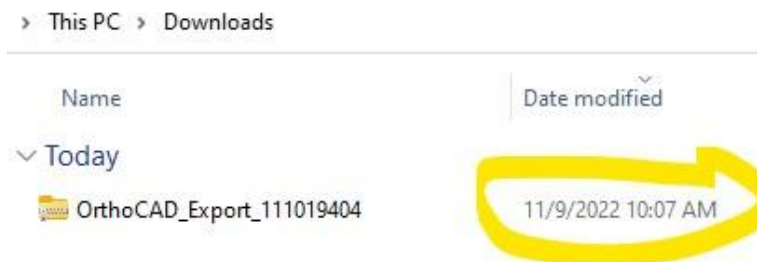

6. Be sure to hit submit

Please contact starazi@cdental.com 510-368-7240 if you have any questions.

By uploading the case you are agreeing to C&M service fees for case preparation of your scans for surgical planning. Fee lists are available upon request.